



**ULTRA FOR MEDICAL PRODUCTS CO.**

## **INSTRUCTION FOR USE**

### **Material Used:**

- P.P. (Polypropylene)
- A.B.S. (Acrylonitrile butadiene styrene)
- Medical grade phthalate free P.V.C. (Poly Vinyl Chloride).
- Master Batch (Blue)
- Master Batch (White)
- H.D.P.E. (High Density Polyethylene)
- Latex Bulb.
- Stainless Steel

### **Indications:**

- The Burette is used to administer Intravenous fluid and medicines into human circulating system by using intravenous catheter or Cannula. The Burette Sets are specially designed to administer measured volume of infusion fluid through gravity.
- I.V. fluid administration.
- I.V. medication administration.
- I.V. nutritional support.

### **Contraindications:**

- Administration of highly viscous fluids.
- Blood transfusion
- Not to be used in patients with known hypersensitivity to any of the materials used.

### **Instruction for Use:**

- Close all the controllers.
- Remove the spike cover and insert the spike firmly into solution container.
- Suspend the solution container with the attached set and open the upper chamber. Close the upper clamp.
- Gently squeeze and release the drip chamber until it is one third full.
- Remove the needle cover & slightly open lower clamp to remove air from the tubing and vein needle. Close the lower clamp.
- Open the upper clamp & allow solution to flow into graduated chamber until the desired volume is obtained. Do not wet air filter by too much solution. Close the upper clamp.
- Make vein puncture.
- Gradually open the lower clamp. Adjust drip rate & control infusion with the clamp. The valve will shut off the flow when solution level in graduated chamber is zero.
- When more solution is required, close the lower clamp & open the upper clamp to fill graduated chamber until desired level. Close the upper clamp and squeeze the drip chamber a little to float the valve up and restart infusion.

### **Warning:**

- The device is Gram-negative bacteria even if the bacteria themselves are killed. For single use only; Reusing can be associated with Cross infection, Device Malfunction and Reactions to endotoxins as sterilization will not inactivate toxins produced by the breakdown of Gram-negative bacteria even if the bacteria themselves are killed.

### **Cautions:**

- Do not use if package is open or damaged.
- Use rubber part only for injecting the medicines.
- Do not use if protective caps loose or missing
- Discard after single use.

### **Instruction For Use**

ISSUE/REV: - 2/0

Page 1 of 2

**Sterile BURETTE SET**

**ULTRA FOR MEDICAL PRODUCTS CO.**

**Conditions of Handling, Preservation and Storage:**

- Not more than 5 cartoons on each other.
- Nice Ventilated place.
- Out of Sun light.
- Room temperature .

ULTRA FOR MEDICAL PRODUCTS CO. DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

**EC REP**

**Obelis<sup>SA</sup>**

53 Bd. Général Wahis  
B-1030 Brussels, Belgium  
Phone: 32.2.732.59.54  
Fax: 32.2.732.60.03  
E-mail: mail@obelis.net  
[www.obelis.net](http://www.obelis.net)



**STERILE EO**

Sterilized by Ethylene Oxide



For Single Use Only



See Instruction for use

**LOT**

Lot No.



Date of Manufacturing



Use by / Expiry date

**REF**

Catalogue No.



Non-Pyrogenic



Don't use if package is not intact



**ISSUE/REV DATE:** 20-12-2020

**Assiut Factory:** Part No. 304, 305, 306, 307, 308, 309, 310, 312, Arab El Awamer- industrial zone, Abnoub, Assiut, Egypt.. **Tel:** 002-088-4964333 (500), 002-088-4964666 (600), 002-088-4964777 (700), 002-088-4964888 (800), 002-088-4964999 (900)

**& Fax:**002-088-4964222**& Mob:** 002-01001558853, 002-01068832355

**Cairo Head Office:** 64, Nakhla El Motaiy Triumph Square Heliopolis **Tel:** 022/4171621-4143794 **& Fax:** 022/4171613 **& Mob:** 01223988200

**Assiut Office:** 23, July Str. **Tel:** 088/2364111 – 2364222 **& Fax:** 088/2334964 **& Mob. :** 01223988202

**Alexandria Office:** 212Abd El salaam Airef Str, Luran **Tel:** 03/5856202– 5856458 **& Fax:** 03/5828988**& Mob:** 01223948666

**E-mail:** [ultra@elaggargroup.com](mailto:ultra@elaggargroup.com), [shady@elaggargroup.com](mailto:shady@elaggargroup.com) **Website:** [www.ultramedumic.com](http://www.ultramedumic.com)

**Instruction For Use**

ISSUE/REV: - 2/0

Page 2 of 2

**Sterile BURETTE SET**